
Directions for use

Ambu® AmbuMan I Ambu® AmbuMan W

TRAINING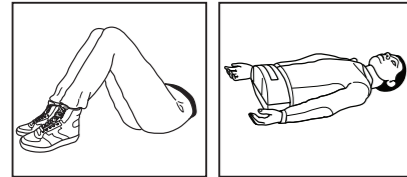

Ambu A/S
Baltorpbakken 13
DK-2750 Ballerup
Denmark
T +45 72 25 20 00
F +45 72 25 20 50
www.ambu.com

492 2340 40 - 04/2011 - V02 Printed in China

English

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Corporate Head Office**& Manufacturer:**

Ambu A/S
Baltorpbakken 13
DK-2750 Ballerup
Denmark
Tel.: +45 7225 2000
Fax: +45 7225 2050
www.ambu.com

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Safety and Regulatory Notices

FCC Statement

This device complies with part 15 of the FCC Rules, Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference and,
- 2) this device must accept any interference received, including interference that may cause undesired operations

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications to the product are not allowed.

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